

Thank you for your order.

For enjoying our product please follow some important indications



Personal Cooling System (PCS): medical device class 1 for multiple sclerosis patients to improve the physical efficiency.

Filling:

1. Remove the white cover of the thin hose at the coolpad.
2. Fill about 30 ml of distilled water in the enclosed injection.
3. Insert the injection into the green end of the hose.
4. Now fill the coolpad slowly and with little pressure. The colour of the coolpad get darker. If the whole coolpad is dark, stop filling. Be careful that not too much water is filled in. Too much water could damage the coolpad. Lock the hose with the white cover again.
5. Once the water evaporates, it can be re-filled as often as needed

To improve the life of the coolpad and to prevent a calcification please use boiled or distilled water.

Washing instructions:

- ~ Our garments are machine washable up to 30° C (gentle cycle only).
- ~ Please use fine-washing detergent
- ~ **Do not** use softener
- ~ Use a washing net bag.
- ~ Do not use an iron and do not tumble dry.

Caution:

The cooling material should not come into contact with sharp objects.

Feedback:

We are interested in a permanent development. Please tell us your opinion: info@unico-swiss-tex.ch

UNICO swiss tex GmbH
Chälengasse 4
CH-6053 Alpnachstad



Tel: +41 (0)41 671 00 71
Fax: +41 (0)41 671 00 70
www.unico-swiss-tex.ch
E-mail: info@unico-swiss-tex.ch